English	INSTRUCTIONS FOR USE
ESPAÑOL	MODO DE EMPLEO
Spanish	
Français	MODE D'EMPLOI
French	
Italiano	ISTRUZIONI PER L'USO
Italian	
Deutsch	GEBRAUCHSANWEISUNG
German	

FKG DENTAIRE
Swiss Dental Products

CH-2304
La Chaux-de-Fonds
Switzerland
www.biorace.ch

English

INSTRUCTIONS FOR USE

The BioRaCe Fill SE™ SOFT RESIN ENDODONTIC OBTURATION SYSTEM

The *BioRaCe Fill* Obturation System has been proven to result in obturations that resist leakage significantly better than gutta percha-based obturations with the added benefit of strengthening the root. The centerpiece of this system is a soft resin obturation material called Resilon®*.

Resilon material is a formulation of polymers of polyester with unique fillers and radiopacifiers in a soft resin matrix. *Resilon* material offers all the features gutta percha provides with the added benefits of enhanced sealing to minimize microleakage and the ability to thermoplasticize at lower temperatures.

The *BioRaCe Fill SE System* with *Resilon* obturation material:

- Is available in points
- Is highly radiopaque
- Looks and handles like gutta percha
- Is fully biocompatible and easily retreatable

BioRaCe Fill SE Self-Etch dual-cure, hydrophilic resin sealer bonds to both *BioRaCe Fill Points* and dentin. An immediate coronal seal is formed by light curing the coronal aspect. The *BioRaCe Fill System* works effectively and seamlessly with all endodontic filling techniques.

Important Notes:

- Complete cleaning and shaping of the canal is essential.
- If calcium hydroxide has been used as an intracanal medicament, completely remove it with EDTA before obturation.

- Always apply additional sealer after downpacking when using the vertical condensation technique to ensure that the sealer completely lines the walls of the canal.
- Use alcohol, instead of chloroform, to remove excess sealer from the chamber floor.
- After light curing the coronal aspect for an immediate coronal seal, place a bonded composite restoration to complete the treatment. *BioRaCe Fill SE Self-Etch Sealer* will self-cure within 45 minutes to seal the entire canal.

Indications:

- The *BioRaCe Fill System* is indicated for the obturation of root canals.

Contraindications:

- *BioRaCe Fill Points* are contraindicated for use with sealers formulated for use with gutta percha.

Storage and Use:

- *BioRaCe Fill Points* – Store and use at room temperature, 60°–80°F (15.5°–26.6°C).
- *BioRaCe Fill SE Self-Etch Sealer* – Refrigeration required. Store at 35°–55°F (2°–12°C). Use at room temperature.

Instructions for Use

1. Irrigate the prepared canal using the following sequence:
 - a. Cleanse the canal with sodium hypochlorite (NaOCl) and dry.
 - b. Remove the smear layer with EDTA.

Note: If EDTA with a surfactant is used, flush the canal with sodium hypochlorite to remove residual surfactant which can interfere with bonding.
 - c. Rinse the canal with sterile water or 2% chlorhexidine and dry.

Important Irrigation Notes: Sodium hypochlorite and peroxide-based lubricants may affect bonding, but are essential for cleaning during the irrigation process. To ensure the best possible bond, please observe the following recommendations:

- a. Sodium hypochlorite (NaOCl) is a required irrigant, but must not be the final irrigant used in the canal.
 - b. Dry the canal with paper points. Do not desiccate the canal with alcohol.
2. Select the appropriate sizes of *BioRaCe Fill Points* for vertical or lateral condensation techniques.
 3. Sterilize the obturation materials. Place *BioRaCe Fill Points* into 2% chlorhexidine or other suitable disinfectant solution for 60 seconds. Rinse with sterile water and dry before use.

Note: Do not use sodium hypochlorite to disinfect Resilon points.
 4. Trial seat *BioRaCe Fill Points* to ensure the selected point(s) reach the working length with tugback. Take a radiograph to confirm apical positioning.
 5. Dispense *BioRaCe Fill SE Self-Etch Sealer* onto a mixing pad.
 6. Apply sealer to the walls of the canal using any of the following techniques:
 - i. A master cone coated with sealer
 - ii. A paper point coated with sealer
 - iii. A small hand file with length control, rotated counter clockwise within the canal
 - iv. A lentulo spiral, 3mm from the apex at 300 RPM.

7. Fill the canal with *BioRaCe Fill Points* using your normal, preferred technique.

Note: If using the vertical condensation technique, apply additional sealer after downpacking. If using lateral condensation, coat each accessory point with sealer.

8. Remove any excess sealer from the chamber floor with alcohol. Do not use chloroform.
9. Take a check radiograph to ensure that the canal is completely filled.
10. Light cure the coronal surface of the obturation for 40 seconds to create an immediate coronal seal. *BioRaCe Fill SE Self-Etch Sealer* will self-cure in the canal within 45 minutes.
11. Seal the access cavity.

Note: Although *BioRaCe Fill* obturations create an immediate coronal seal when light cured and seal the canal within 45 minutes upon self-cure, the placement of a bonded composite over the pulpal floor is recommended. The patient should be encouraged to receive the final restoration as soon as possible. The quality of the restoration is a significant factor in the success of the endodontic treatment.

Backfilling with Heated Delivery Systems

If using System B™** to backfill the obturation, set the temperature to 150°C and the power to 10.

** System B is a trademark of Sybron Dental Specialties.

For information on *BioRaCe Fill Points* and *Self-Etch Sealer* visit www.biorace.ch or call +41 (0)32 924 22 44. Points are available in 50 pack refills for your convenience.

Limited Warranty

Express Limited Warranty:

FKG DENTAIRE warrants that this product is not defective for a period of one year.

Buyer's Exclusive Remedy:

In the event that this product proves to be defective within a period of one (1) year from the date of shipment to Buyer, Buyer's exclusive remedy shall be at FKG DENTAIRE's option of either replacement of the product or defective part or component, free of charge, or refund of the purchase price. Buyer is only entitled to this remedy upon showing that the product has been stored and used in accordance with FKG DENTAIRE's instructions, with the exception that this warranty nevertheless extends to products whose expiration dates fall within the one-year limited warranty period.

Disclaimer of Consequential Damages:

FKG DENTAIRE hereby disclaims any liability for loss or damage, direct or consequential, that result from the use of this product.

Disclaimer of All Other Warranties:

THERE ARE NO OTHER WARRANTIES OF ANY KIND, EXPRESS OR IMPLIED, CONNECTED WITH THE SALE OF THIS PRODUCT. FKG DENTAIRE SPECIFICALLY DISCLAIMS ANY IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE.

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*Resilon ® is a registered trademark of Pentron Corporation.